

Quality For Life



MK2218 SERVICE MANUAL

Read Instruction Before Using

TABLE OF CONTENT

1. Feature

2. Warning

3. Measurement

4. Assemble

5. Function

6. Warranty and Maintenance

1. FEATURE

A. Canopy

B. Headrest

C. Recline seat back

D. Abductor

E. Footrest supporter

F. Adjustable footrest

G. Front swivel wheel

H. Adjustable handle

I. 5-Point harness

J. Footrest angle adjustment

K. Seat release lever


L. Brakes

M. Rear wheel

N. Basket

O. Folding latch

P. Seat recline lever

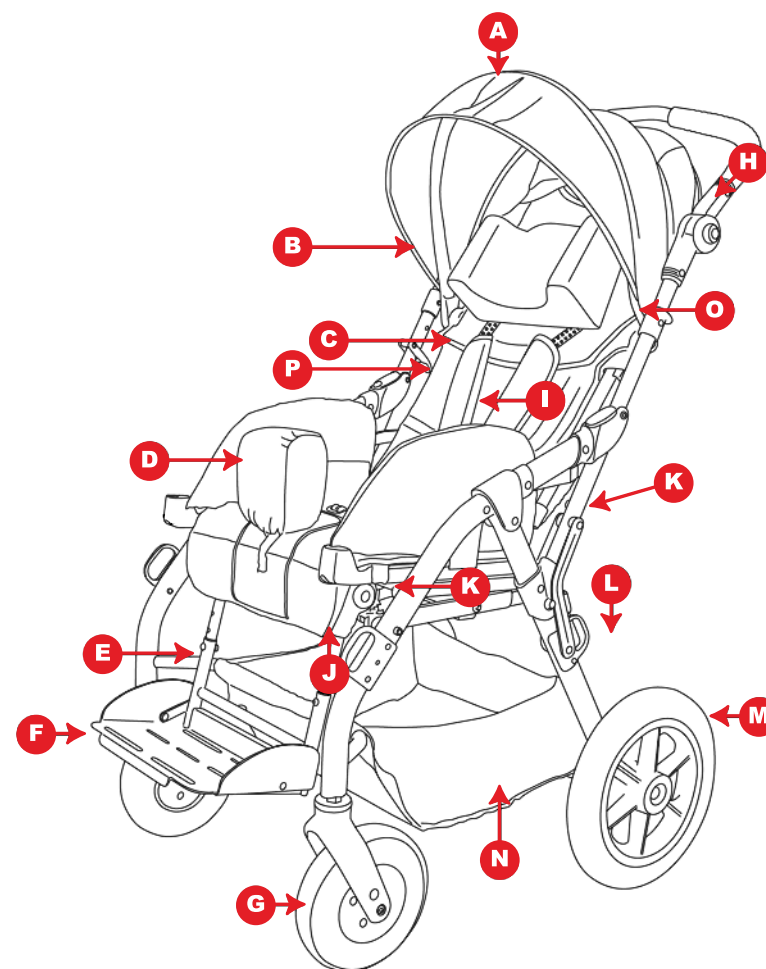


WARNING

READ AND FOLLOW ALL WARNINGS AND INSTRUCTIONS!
FAILURE TO DO SO COULD RESULT IN SERIOUS INJURY OR DEATH!


- **NEVER** leave child unattended. **ALWAYS** keep child in view while in the stroller.
- Avoid serious injury from falling or sliding out. **ALWAYS** use seat belt.
- When making adjustments to the stroller, **ALWAYS** ensure that all parts of the child's body are clear of any moving parts of the stroller.
- **DO NOT** use stroller if any parts are missing or broken.
- **DO NOT** allow child to climb on or around stroller.
- **DO NOT** allow child to stand on the footrest. stroller may tip over and injure child.
- **NEVER** leave stroller on a hill or incline, even with brakes set, stroller may tip injuring the child.
- To avoid accidental rolling, **ALWAYS** set brakes when stroller is not being pushed.

3. MEASUREMENT


Product Code	PC2218 (Mfg.MK2218)	
F/R Wheel	7.5" / 12"	
Rear Wheel	Fixed	Quick Release
Overall Width (W)	650mm	655mm
Overall Height (H)	950~1180mm	
Overall Length (L)	860mm	
Folding L*H*W	450*655*970mm	
Footrest Length (FL)	260~345mm	
Seat Width (SW)	200~375mm	
Seat Depth (SD)	280~340mm	
Back Height (BH)	450~650mm	
Tilt-In-Space (Seat)	-30° ~ +30°	
Recline Angle (Seat)	55° ~ 175°	
Product Weight	19.9~23kgs	
Weight Capacity	60kgs	

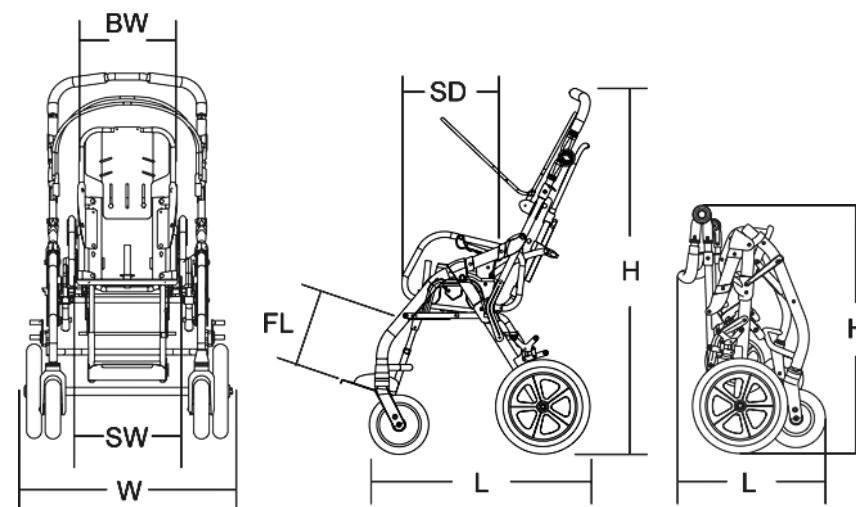


3.1 ACCESSORY

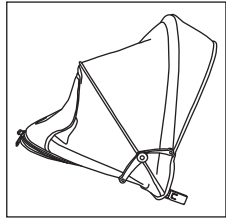


Canopy

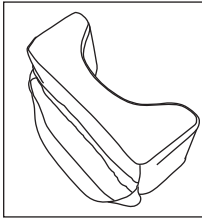


Headrest

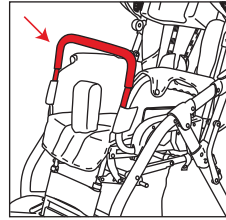


Grab rail

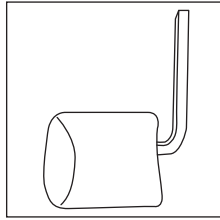


Abductor

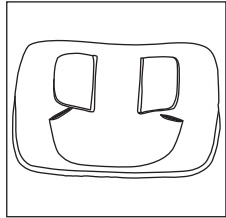


Headrest pad

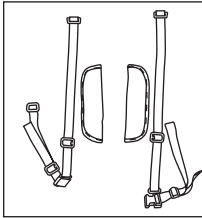


Harness

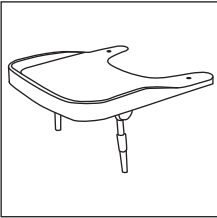


Tray

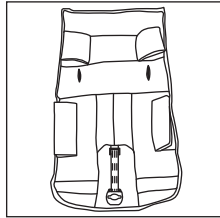


Seat pad

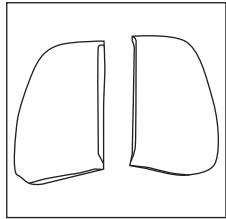


Armrest sleeve

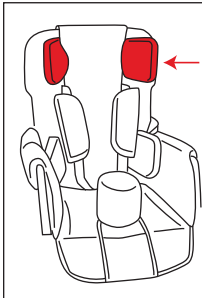


Head support

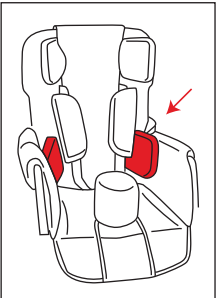


Trunk support

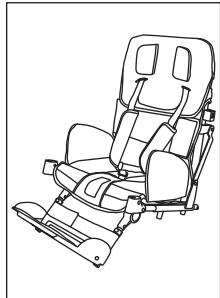

Seat unit

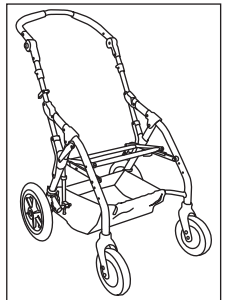




Mobility Base

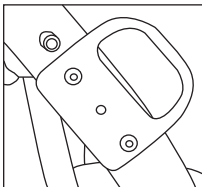



Tie-down kit

4. ASSEMBLE

4.1 UNFOLD PUSHCHAIR

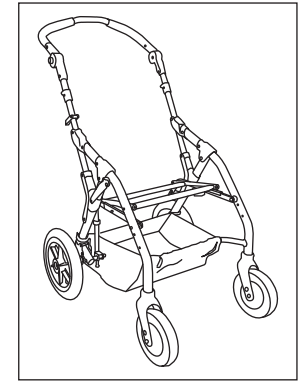
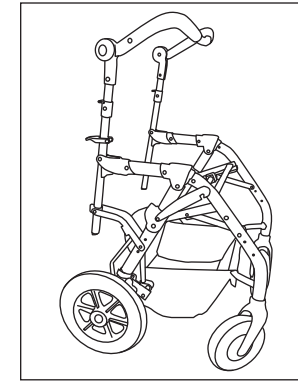
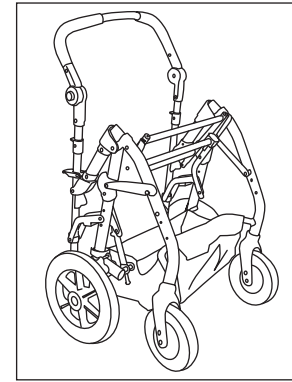


4.2 SEAT UNIT FIT ON

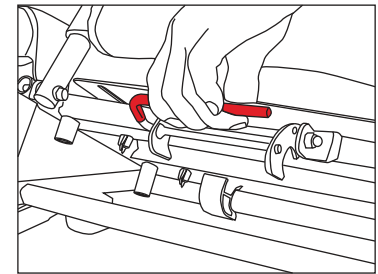
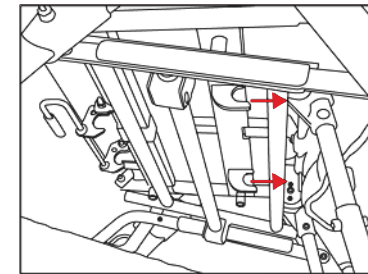


WARNING: Make sure the seat unit is well fitted on the mobility base.

4.3 CANOPY ASSEMBLE

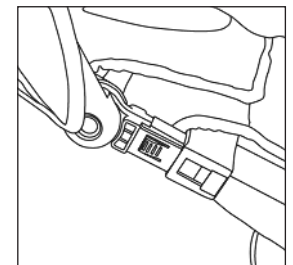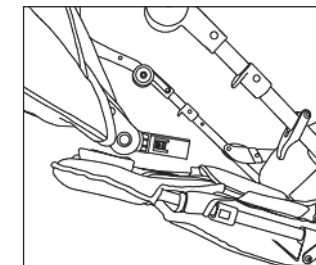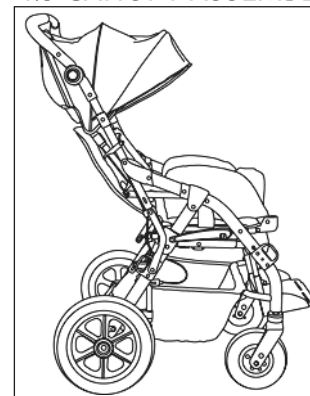

4.4 FOLDING STROLLER & SEAT ANGLE ADJUSTMENT

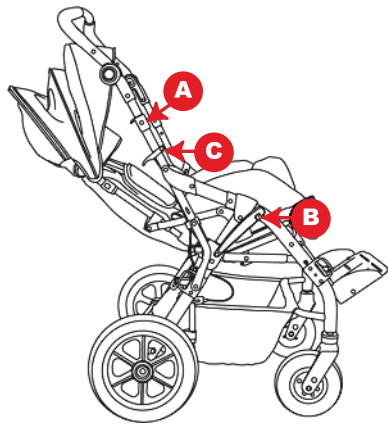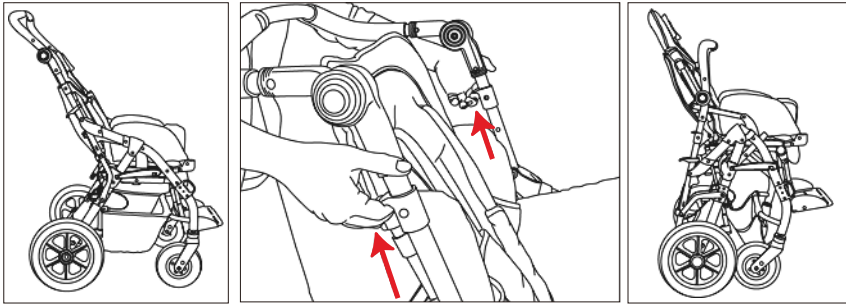

To fold stroller, lock rear brakes and remove your baby.


Close canopy if canopy is assembled on the stroller.

Pull up the fold latch (A), and push stroller backward until it's in folded position.


Lock the folding hook (B)

Pull up the recline lever (C) to adjust the seat angle.


4.5 SEAT PAD ASSEMBLE

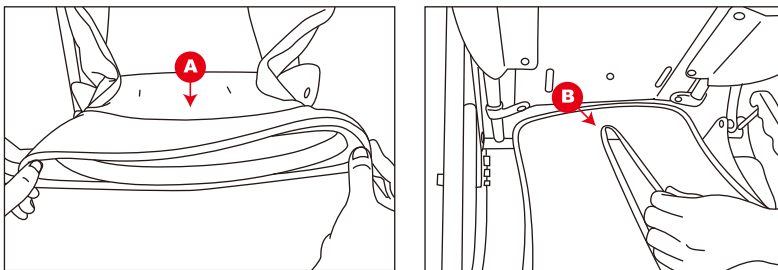






Place the seat pad into the seat back (A) and the back cover to go through back rest bottom end (B).

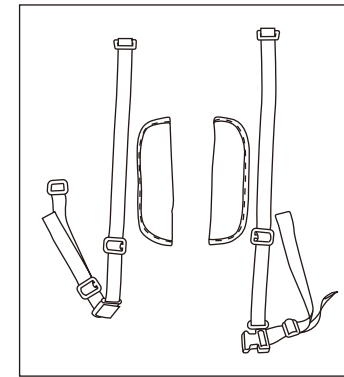

Chest pad (C) onto the strap.

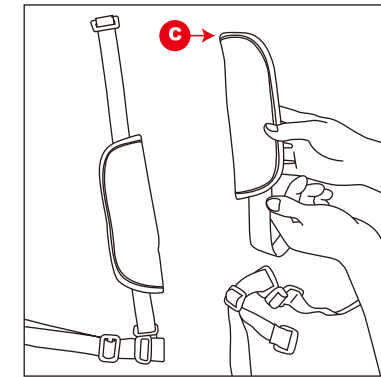


4.6 ASSEMBLE HARNESS

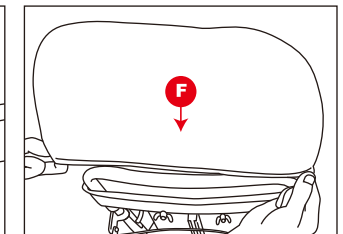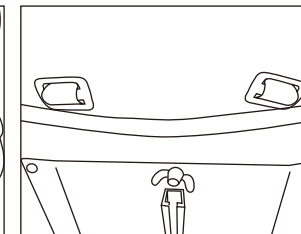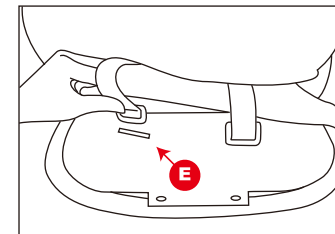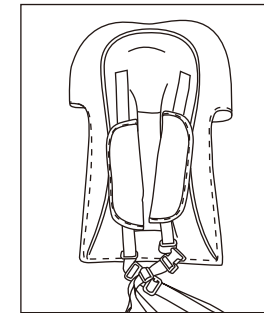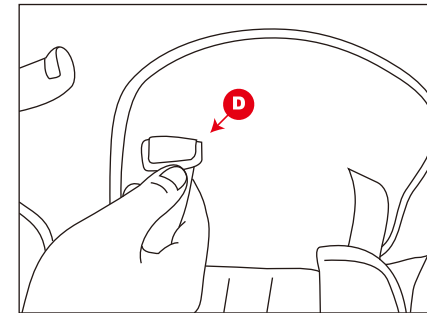

Thread strap into both the seat pad (D) and the backrest slot (E).

Cover the back rest with backrest pad as (F).

WARNING: Misthreading straps and buckles can lead to straps slipping and the child not being properly restrained.

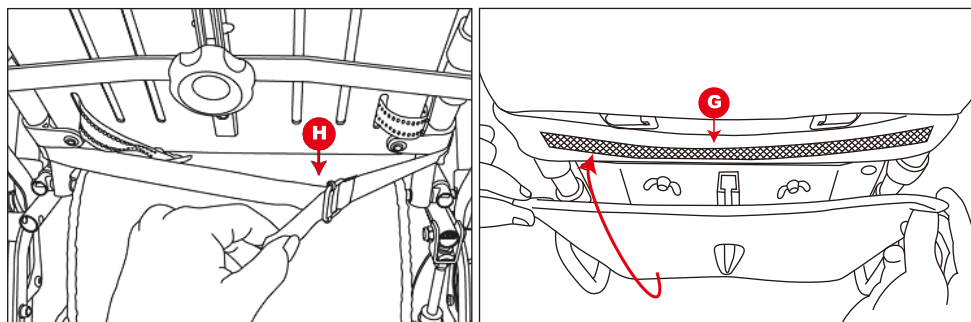


Buckle (H) up.

Pull up the seat flap and fasten Velcro on the back (G).


5. FUNCTION

5.1 HANDLE ADJUSTMENT

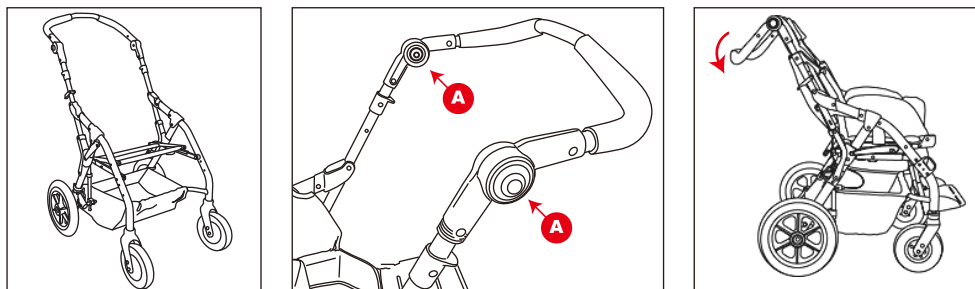

To adjust the handle angle, push the adjustment button (A).


5.2 CANOPY ADJUSTMENT

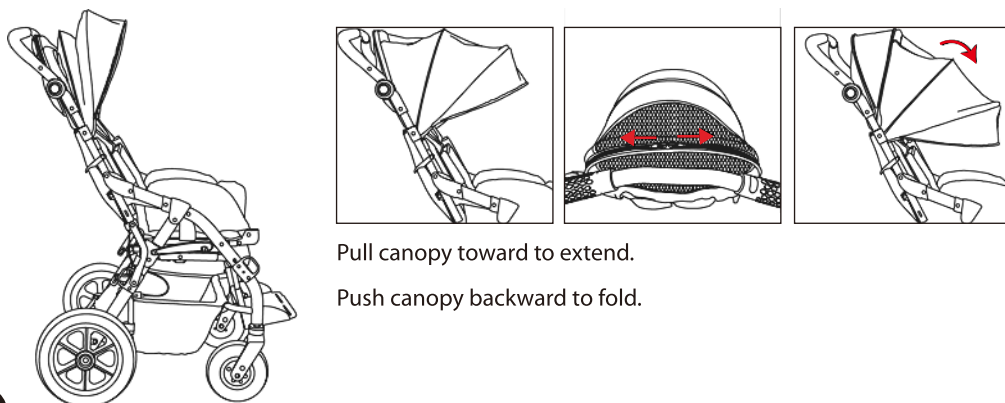

Pull canopy toward to extend.

Push canopy backward to fold.

5.3 REVERSE SEAT

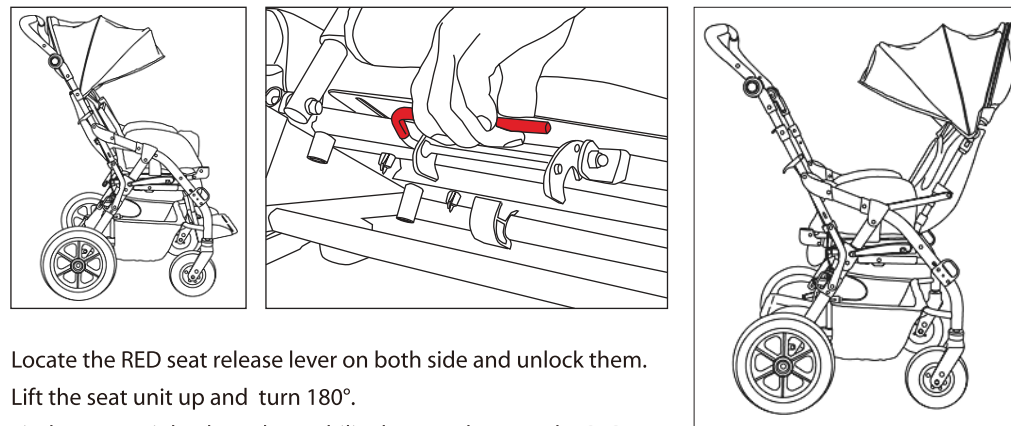


Locate the RED seat release lever on both side and unlock them.

Lift the seat unit up and turn 180°.

Fit the seat unit back on the mobility base and secure the RED seat release lever.

WARNING: REMOVE the child before reverse seat.


5.4 RECLINE

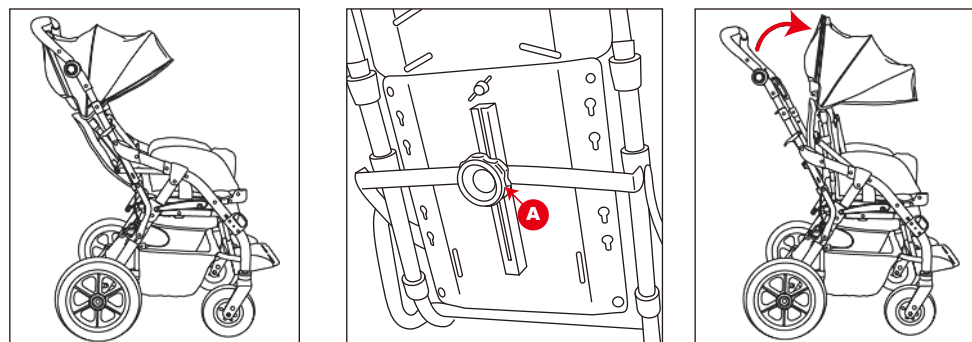

Turn the adjustment grip (A) to recline back rest.


WARNING: Please always hold onto the back rest when recline or raise.

5.5 BACK HEIGHT ADJUSTMENT

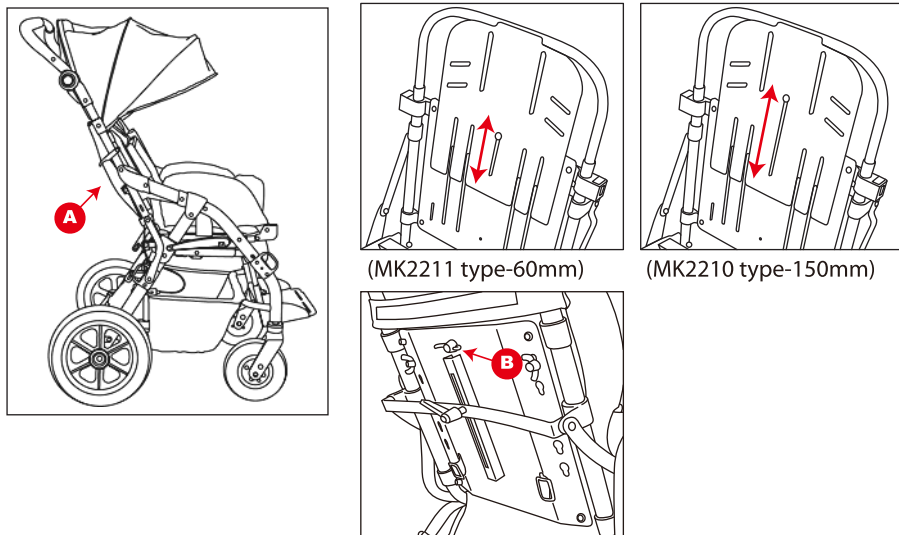

Remove pad (A) and loose screw (B) to adjust the back height.


5.6 FASTEN SEAT BELT AND ABDUCTOR

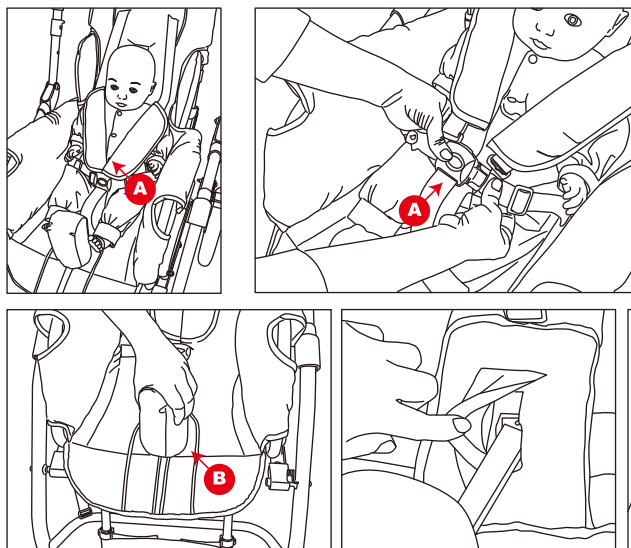

Remove abductor (B) and seat your child, fasten seat belt as (A). Insert abductor back on again between child's legs, fasten the screw under the seat.

**WARNING: Avoid serious injury from falling or sliding out.
Always use seat belt.**


5.7 SEAT WIDTH AND DEPTH ADJUSTMENT

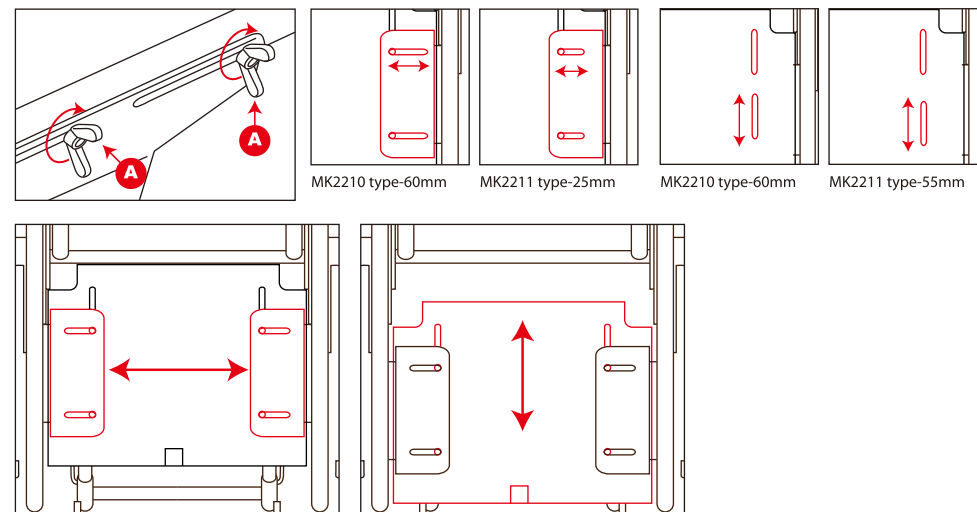

Remove seat pad and loose the screws (A) under the seat, this will allow you to adjust seat width and depth.


WARNING: REMOVE the child before seat adjustment.

5.8 HEADREST FIT ON

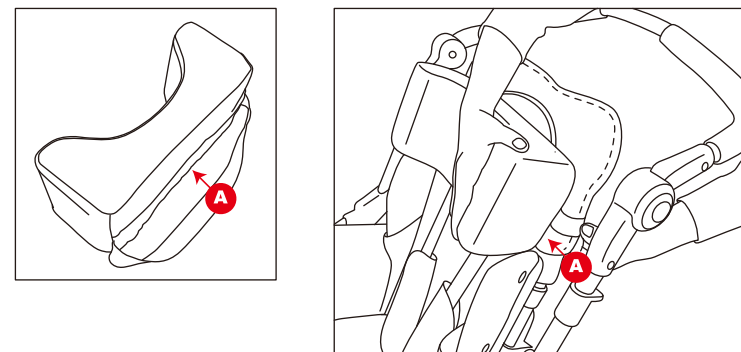


Fit headrest (A) on the seat back as picture shown.

5.9 FOOTREST ANGLE AND LENGTH ADJUSTMENT

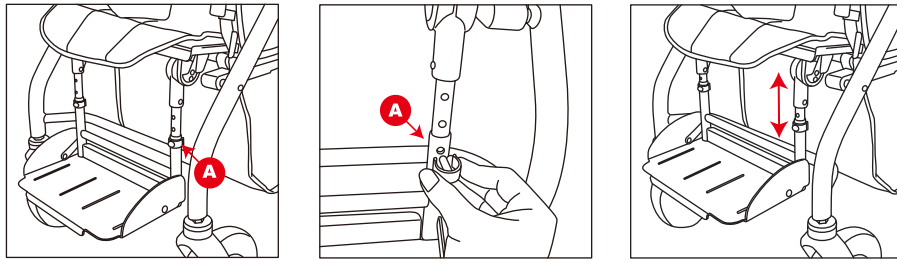


Take out clip (A) from both sides and adjust the height position.

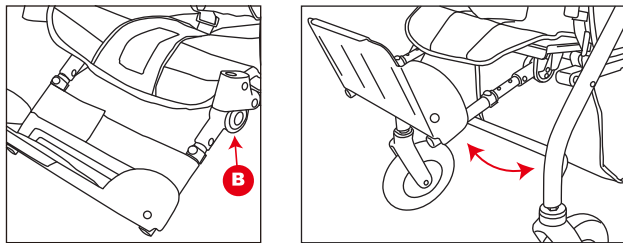

Press button (B) to adjust footrest angle.

WARNING: DO NOT USE THE FOOTREST AS TREADLE.

5.10 BRAKE

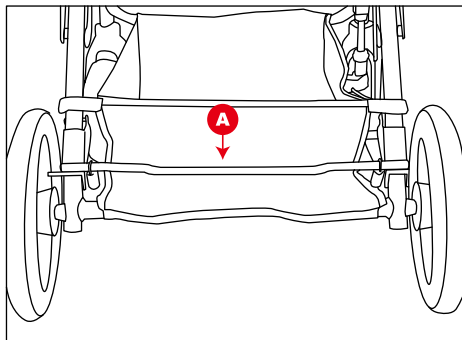

Step on both rear brakes (A) to lock the stroller.

6. WARRANTY AND MAINTENANCE

There is a comprehensive twelve-month warranty from the date on which your new stroller is delivered. The warranty covers the stroller for repairs or replacement during this period. For more detail, please see the warranty conditions:

1. Any work or replacement part installation must be carried out by an authorized dealer / service agent.
2. To apply the warranty should your stroller require attention please contact the outlet from which you purchased the stroller.
3. Should any part of the stroller require repair or full or part replacement, as a result of a manufacturing or material defect within the warranty period, parts will be supplied free of charge. Note: The guarantee is not transferable.
4. Any repaired or replaced parts will be covered by the balance of the warranty period on the stroller.
5. Parts replaced after the original warranty has expired will be covered by a three-month warranty.
6. Consumable items supplied will not generally be covered during the normal warranty period unless such items require repair or replacement clearly as a direct result of a manufacturing or material defect. Such items include (among others): upholstery and tyres.
7. The above warranty conditions apply to brand new stroller. Second-hand strollers supplied directly by Rehab Product Company Limited carry a six-month warranty period. If you are unsure whether your stroller is covered contact your dealer.
8. Under normal circumstances, no responsibility will be accepted where the stroller has required assistance as a direct result of:
 - a. the stroller part not having been maintained in accordance with the manufacturers recommendations
 - b. failure to use the manufacturer's specified parts
 - c. the stroller or part having been damaged due to neglect, accident or improper use
 - d. the stroller or part having been altered from the manufacturer's specification or repairs having been attempted before the dealer is notified

The manufacturer reserves the right to alter without notice any weights, measurements or other technical data. All figures, measurements and capacities are approximate and do not constitute specifications.